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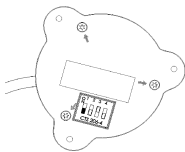
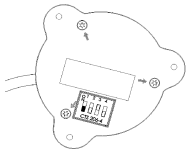
Presentation of PapooSwitch®

PapooSwitch® is a high-sensitivity tactile switch. PapooSwitch® detects the presence of the human body and acts like a switch.

Standard Operating

Your PapooSwitch® is ready for use. Simply turn it on :

1. Slide the switch #1 (located under the device) to ON.
2. Connect PapooSwitch® to the device to be driven, using the cable.
3. Tap the top of PapooSwitch® to make it work.

Customizable functioning

Several functionalities allow you to adapt the PapooSwitch to its user.

1. High sensitivity

The PapooSwitch can detect a finger at 1cm, thanks to its high sensitivity electronics.

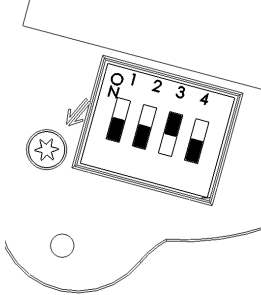
2. Sound notification

A beep can be emitted when the switch is touched.

3. Light notification

A light can be emitted when the switch is touched.

You can activate those functions using the toggle switches located under your PapooSwitch :

switch	parameter	action
1	Power	On / Off
2	Light	On / Off
3	Sound	On / Off
4	High Sensitivity	On / Off

Factory settings :

Light :	Off
Power :	Off
Beep :	On
High sensitivity :	Off

Light indicator

The PapooSwitch® has a light indicator that lets you know its battery level

Battery level summary table

Color	Battery state
Red	Full battery
Blinking Orange	Low battery

Replace as soon as possible if the battery indicator blink orange.

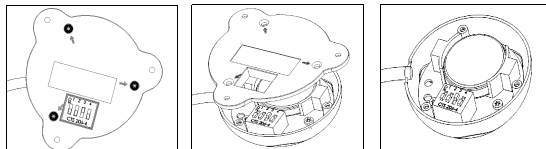
The red light only appears when a touch is detected depending on your settings.

Changing the battery

The PapooSwitch® uses a CR2450 type button battery. SMARTIO recommends Renata batteries.

This battery is available at www.papoo.fr

Step 1

Step 2

Step 3

Step 1 - Unscrew the 3 screws located under the contactor with a cross-shaped screwdriver.

Step 2 - Remove the contactor lower part.

Step 3 - Change the battery.

Step 4 - Put the lower part back in place. Ensure that the cable is not caught between the two halves of the contactor : it must fit in the notch located on the lower part.

Step 5 - Screw the lower part back on using the 3 screws.

Don't throw any worn battery in household garbage containers !

PapooSwitch

(non-contractual picture)

Technical characteristics

Dimensions	40 x 15 mm
Weight	Less than 50 gr.
Battery	Lithium – manganese oxide CR2450 Renata
Tension	3 V
Jack	Mono Diam 3,5 mm

Troubleshooting

Should you encounter any problem using your PapooSwitch®, please refer to the table below. If you can't solve your problem, go to www.papoo.fr for more help.

Issue	Solutions
The PapooSwitch® does not light up when plugged to a device.	- Toggle the switch #2 to its ON (upper) position. - Check the battery level. - Change the battery.
The PapooSwitch® does not emit any sound	- Toggle the switch #3 to its ON (upper) position.
The device does not detect the PapooSwitch®.	- Unplug and plug the cable from the device. - Ensure that the cable is plugged all the way in.

Note : For further information, please contact your provider.

Safety instructions

- ✓ Carefully read the given instructions
- ✓ Have reparations performed only by a qualified technician. Do not try to open the switch for purposes other than battery change.
- ✓ Do not clean your PapooSwitch® with any solvent, diluent or spirit. Use a wet cloth instead.
- ✓ Do not keep the PapooSwitch® in wet or high temperature places.
- ✓ Keep your PapooSwitch® away from dirt, mud, sand, dust, water, toxic gases or salt : those elements could damage it.
- ✓ Do not throw your PapooSwitch® in household garbage containers. Send it back to your dealer.
- ✓ Not suited for children under 36 months : contains small parts that can be ingested.

Norm compliance

This product complies with the 93/42/CEE directive, transcribed in the national right by the 92-292 decree on the 16th of March 1995. Class I device according to the first article of the aforesaid directive. CE produced and certified.

Warranty conditions

To know the applicable warranty conditions to your device, please contact your provider.